COLCHON DE VACIO

VACUUM MATTRESSES

PA-91

PA-187

SERIE 3000:

Descripción

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- PA-187

100 X 205

- PA-91

80 X 210

OPCIONAL:

3001B BOMBA DE VACIO

1610 BOLSA TRANSPORTE
COLCHON VACIO

GARANTIA

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LIMITATION OF LIABILITY

The warranty for this product is applicable for repairs, when the damages are not caused by improper handling or abuse or neglect by the user. Shipping charges for transport for repair are not covered. Product repair or refund of the amount will only be performed if the total damage and returning the defective FESA. In case of attempted repair by the customer the distributor not responsible for the arrangement and will be out of warranty.

WARRANTY PERIOD

PROMEBA ensures that your product is free from defects in material and workmanship. The warranty period is 2 years from date of shipment.

These models are designed to aid in the immobilization of patients with possible injuries or fractures.

Weight 6.65 kg

Max. load 150kg

IMPORTANT

- **The vacuum mattress is only for professional use**
- **Use it following the instructions in this manual**
- **Practice using the mattress before putting into service**
- **Must be used by at least two well-qualified workers, which may take up to four when transporting a patient requires it.**
- **If you pass the vacuum mattress include this manual for track maintenance.**
- **In case of lifting loads weighing more than 150kg., It is not possible using only the vacuum mattress, since it ensures good stability and a deformable support.**
- **Before any move to ensure the patient with quick release belt.**
- **Do not leave the patient without health care while being transported on the mattress.**

MAINTENANCE

Wash the mattress empty after each use. This will avoid cross-infection by secretions or debris. Cleaning is done with warm water and mild detergent, dry with a soft cloth. Disinfected with an appropriate product for blood (if necessary).

USE THE MATTRESS

Unpack the product and check all components received. If the mattress is damaged in shipping or save the package box and contact your dealer within 48 hours.


Staff must be qualified to determine the procedure and equipment appropriate to use.

There will be a minimum of 2 operators for the use of the mattress may

be needed to reach

4.

Check-uniformity of the granules inside the mattress, if it is over that of an unstable or irregularities, use of the mattress above the transportation table.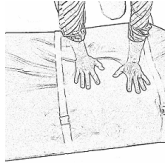

-Place the casualties, rescue operations or using a table spoon. Placing the patient's feet to the height of the vacuum tube, leaving it in a manner which is easily accessible by the operator.


-Once in place the injured shape the mattress according to patient silhouette and shape suitable for immobilization, blocking any inertial motion. Check the immobilization of all the main points, head, shoulders, pelvis and lower limbs.

- He advises the application of a weight on the end of the mattress where you head this way for spinal decompression will not suffer pressure.

- Fasten the quick release fasteners, and pull the tabs on the tape so that they fit as possible to the contour of the mattress.

- Open the valve and connect vacuum aspirator tube secretions or manual vacuum pump. After reaching the desired degree of relief to close the valve.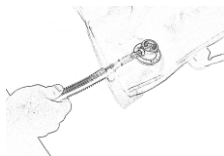

-Once the mattress is ready for transport, check both the quick release straps as barbuquejos to continue tight.

COVER REMOVAL

This series differs for convenience in being able to separate the internal chamber of the outer sleeve. While this is very useful when cleaning, replacement or repair.

-Open the outer sheath by moving the slide zipper from head to toe.

-Unscrew connector and washer.

WASHER

-Extract the mattress inside the sheath for repair.