


SPINE BOARD AND SCOOP STRETCHER

PA-179

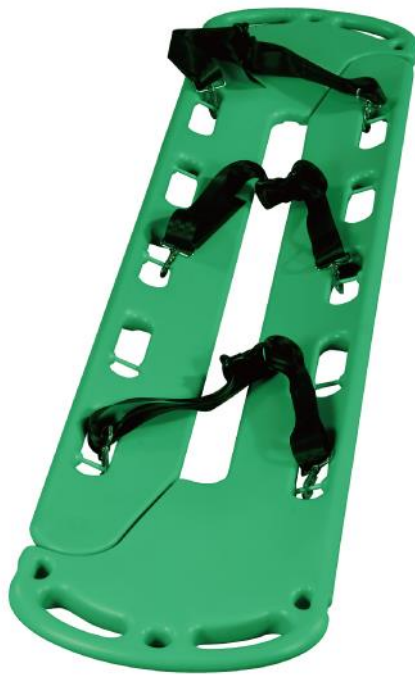

Instructions Manual

1. Introduction about the manual

What described here are parts of the manual, please read the introduction carefully as they provide important information relate to the safety use and maintenance procedures. The description and pictures in the manual are not binding, “PROMEBA” has the right to modify any information to update the product’s quality in this manual. The graph and pictures are only examples and may be further improved.

2. Applicable Scope

It is mainly used for hospitals, emergency centers, sports, ambulance and battle fields carrying patients and wounded person.

3. Features

1. It can be used as a scoop stretcher or a full spine board.
2. Equipped with a clutch hinge in the middle of both ends, which can separate the stretcher into two parts.
3. The stretcher could strengthen the patients at origin place which could reduce the further hurt to the patients.
4. The patient can be quickly & easily put in the stretcher and also the stretcher can be taken out from the back of patient without moving the patient.
5. Can be used together with head immobilize. Allow X-ray examination.
6. Suitable for patients with suspected hip, pelvic, and spinal injuries.
7. Shell: High density polyethylene.
- 8. Filler: polyurethane foam.
9. Latches: Aluminum, Stainless steel.

4. Technical parameter

Model	PA-179
Unfolded Size(L×W×H)	187×42×6cm
Load Bearing	204kg

5. Use Method

1. Use the thumb to pull the clutch buttons in both ends of the stretcher, which can separate the stretcher into two parts. (Referred to the part as shown in Figure 1)

Warning: When the clutch buttons are not in the lock status, carrying and transporting the patients are forbidden.



2. Put the lateral plates directly from both sides of the patients and fix the clutch hinge.
3. Wear the safety belt and tension it which could reduce the further hurt to the patients because of sliding.

6. Cautions

1. The operating persons shall practice the use of this stretcher and possess first-aid skills. The operating persons must operate it in accordance with the introduction during the use of stretcher. Otherwise, our company shall not be responsible for the dangers occurred.
2. When transporting the patients, be sure the clutch buttons at both ends are locked, and use safety belts to ensure safety.
3. When adjusting the length, be sure the position is locked, please do not use non-locked stretcher so as not to hurt patients. Don't adjust the length too long, choose the proper position during the use of the stretcher.

7. Maintenance

1. Keep clean as routine (including disinfect).
2. Always check if the accessories are loosed or not.

8. Storage

1. Store this stretcher in the place of damp proof and non-corrosion environment.
2. When in storage and transportation, stack it as shown in the packing.

9. Transport

1. Be transported by air, sea and land.
2. The stretcher may be carried by manual.



10. Lists of Accessories

No.	Name	No. of pieces
1	Safety belt	3
2	Introduction	1
3	Qualified Certificate	1